



G-Force Configuration Guide – Windows 10 version 1903


Right click in the G-Force visualizer. Under Audio Input Source, select Stereo Out (SoundSpectrum Audio Cable).

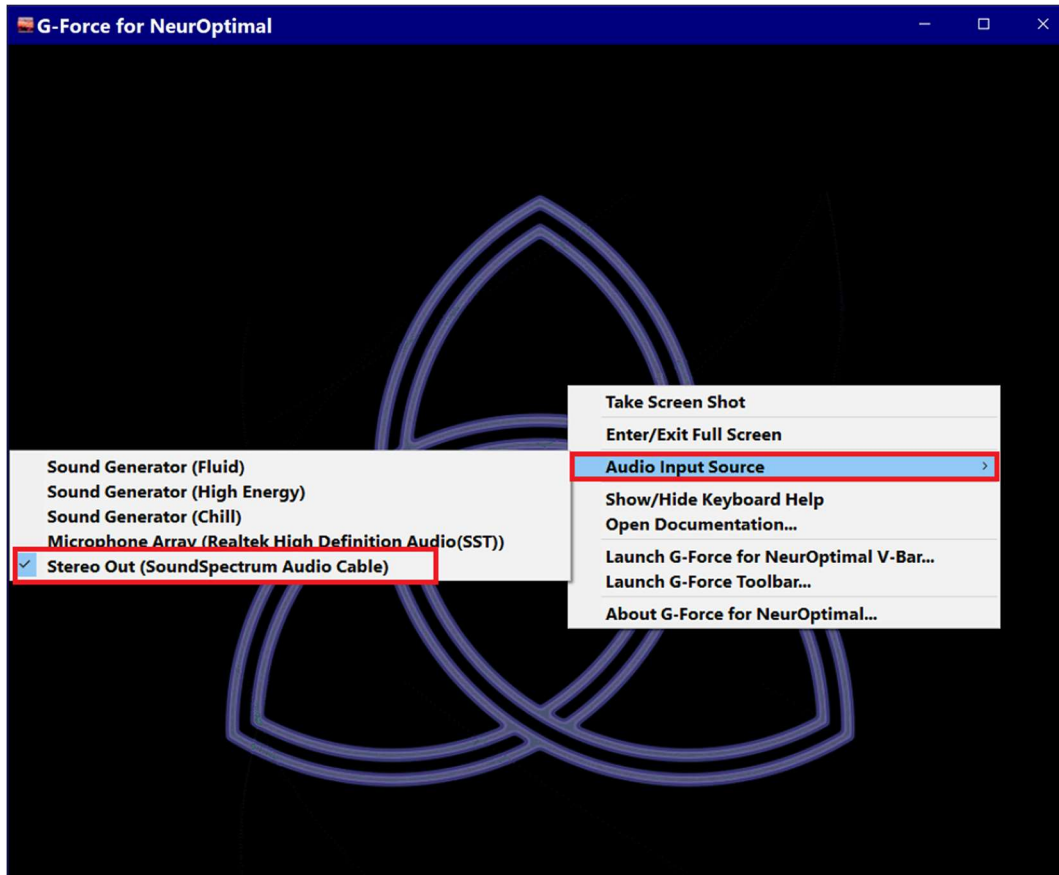


Right click the audio icon on the taskbar; Open Sound settings.

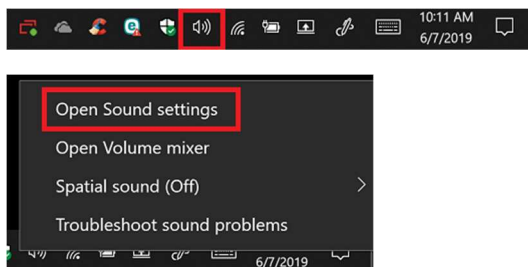


From the Sound settings menu, open the Sound Control Panel.

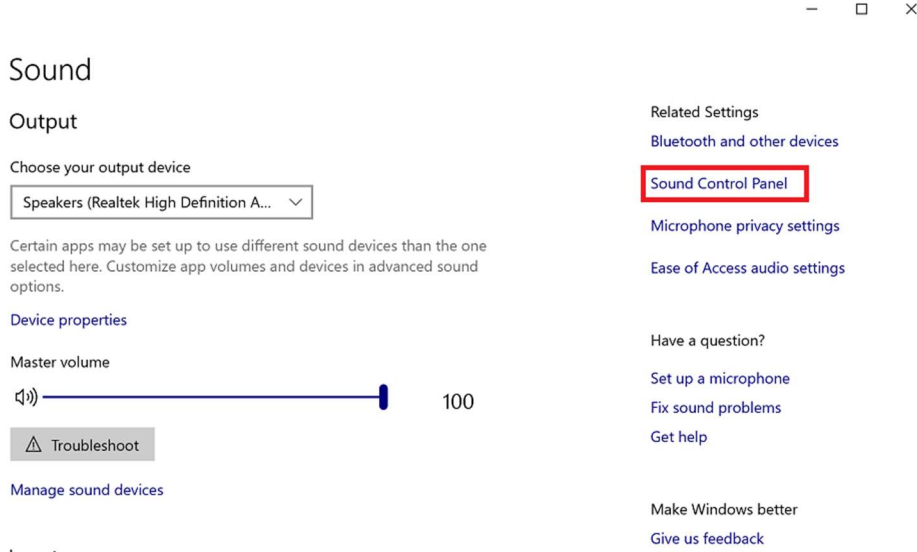


Speakers should be set as the default Playback device.

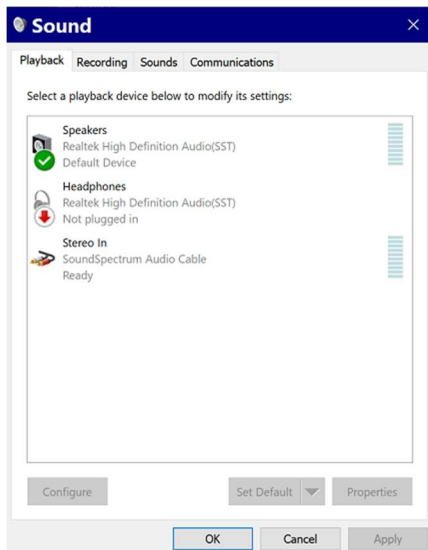


Open Stereo Out – SoundSpectrum Audio Cable.

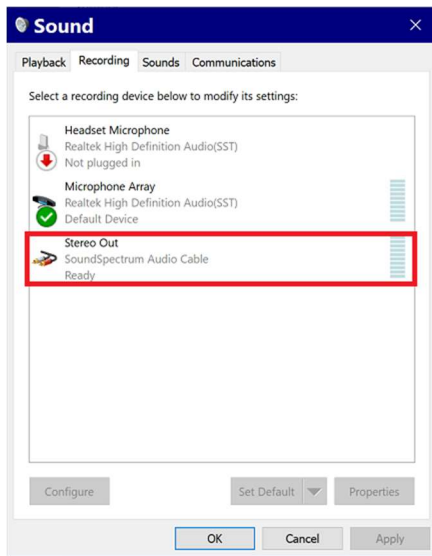

Select “Listen to this device”.

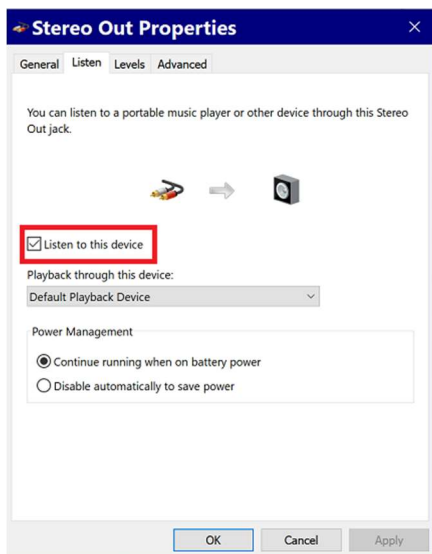

Right click anywhere on the taskbar. Open Taskbar settings.

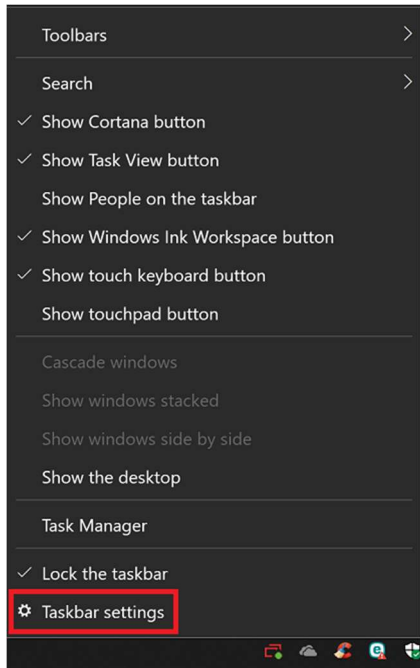

In Taskbar settings, select “Turn system icons on or off”.

Taskbar

Lock the taskbar



Automatically hide the taskbar in desktop mode



Automatically hide the taskbar in tablet mode



Use small taskbar buttons



Use Peek to preview the desktop when you move your mouse to the Show desktop button at the end of the taskbar



Replace Command Prompt with Windows PowerShell in the menu when I right-click the start button or press Windows key+X



Show badges on taskbar buttons



Taskbar location on screen



Combine taskbar buttons



[How do I customize taskbars?](#)

Notification area

Select which icons appear on the taskbar

[Turn system icons on or off](#)

Turn off the Microphone here. It is usually at the bottom of the list depending on your hardware.



Microphone

