




Disk Cleanup Guide – Windows 10 version 2004

Open the Start Panel. Right click Disk Cleanup and run it as as administrator.

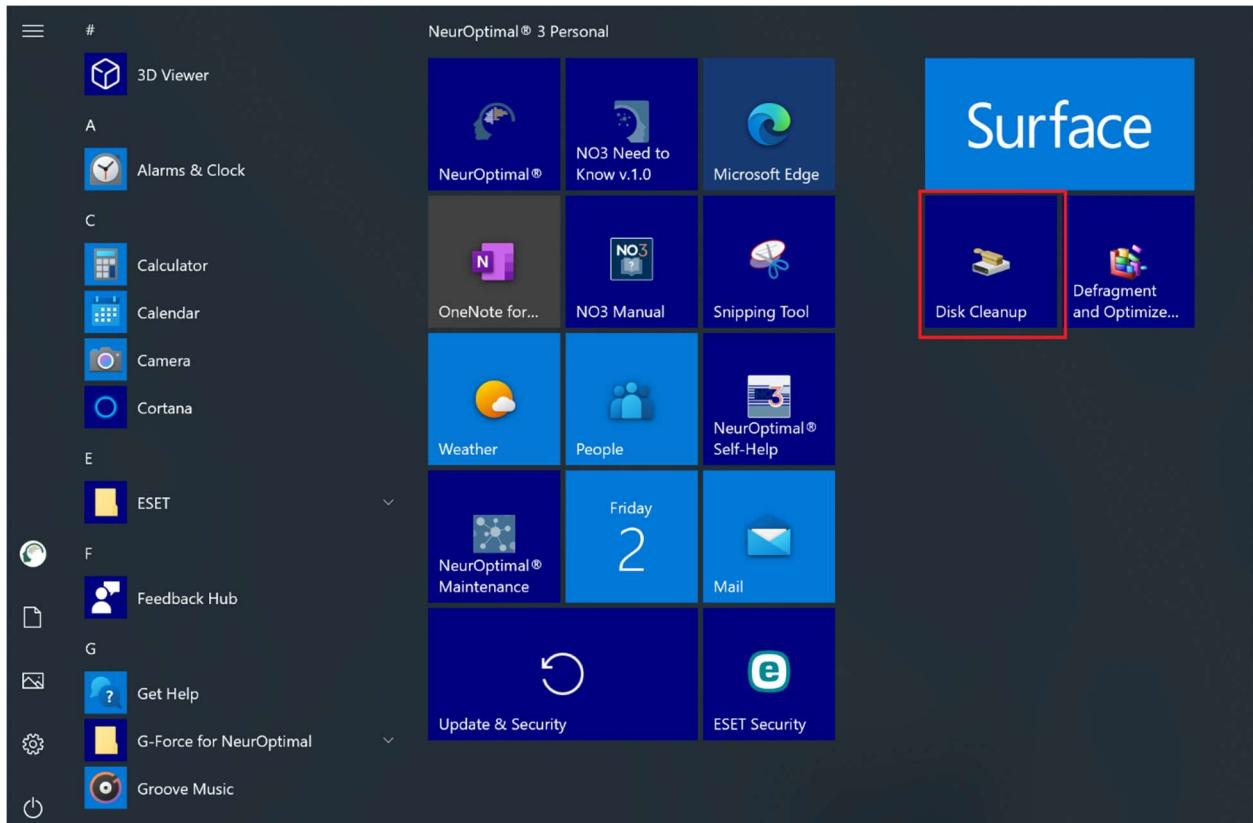

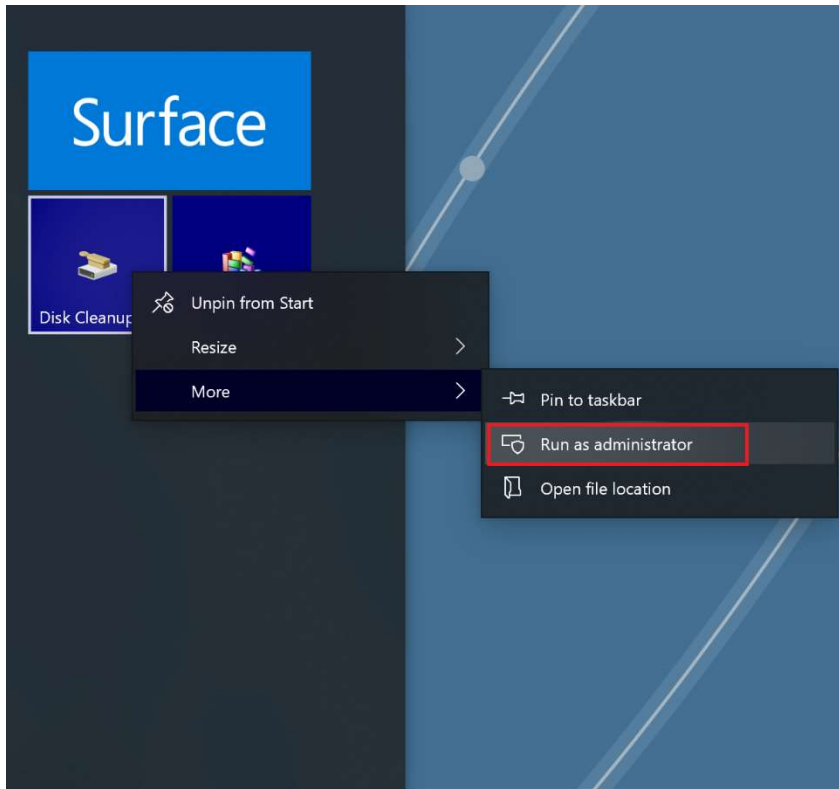

Select OS (C:). PHOENIX (D:) is the Zengar recovery partition and can be ignored.

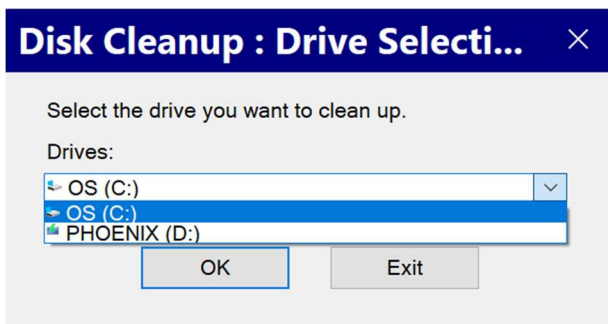

Select all items in the list (pictured below).

Disk Cleanup for OS (C:)



Disk Cleanup **More Options**



You can use Disk Cleanup to free up to 182 MB of disk space on OS (C:).

Files to delete:

<input checked="" type="checkbox"/>	Microsoft Defender Antivirus	5.93 MB	^
<input checked="" type="checkbox"/>	Downloaded Program Files	0 bytes	
<input checked="" type="checkbox"/>	Temporary Internet Files	1.95 MB	
<input checked="" type="checkbox"/>	Windows error reports and feedback dia...	44.2 KB	
<input checked="" type="checkbox"/>	DirectX Shader Cache	0 bytes	v

Total amount of disk space you gain: 182 MB

Description

Non critical files used by Microsoft Defender Antivirus

OK

Cancel

Disk Cleanup for OS (C:)



Disk Cleanup **More Options**



You can use Disk Cleanup to free up to 182 MB of disk space on OS (C:).

Files to delete:

<input checked="" type="checkbox"/>	Delivery Optimization Files	156 MB	^
<input checked="" type="checkbox"/>	Device driver packages	0 bytes	
<input checked="" type="checkbox"/>	Language Resource Files	0 bytes	
<input checked="" type="checkbox"/>	Recycle Bin	0 bytes	
<input checked="" type="checkbox"/>	Temporary files	144 KB	v

Total amount of disk space you gain: 182 MB

Description

Non critical files used by Microsoft Defender Antivirus

OK

Cancel

Disk Cleanup for OS (C:)



Disk Cleanup **More Options**



You can use Disk Cleanup to free up to 182 MB of disk space on OS (C:).

Files to delete:

<input checked="" type="checkbox"/>	Device driver packages	0 bytes	^
<input checked="" type="checkbox"/>	Language Resource Files	0 bytes	
<input checked="" type="checkbox"/>	Recycle Bin	0 bytes	
<input checked="" type="checkbox"/>	Temporary files	144 KB	
<input checked="" type="checkbox"/>	Thumbnails	18.0 MB	v

Total amount of disk space you gain: 182 MB

Description

Non critical files used by Microsoft Defender Antivirus

OK

Cancel

After selecting all items, open the “More Options” tab. Click “Clean up...”. Then press OK. Disk Cleanup will now remove all the files selected and delete old restore points.

