

How to start a NeurOptimal® Session

1

Identify items.

Paste



zAmp & USB cable



Tablet/Computer




2

Launch NeurOptimal®.

Launch NeurOptimal® by tapping on the NeurOptimal® icon in the Task Bar at the bottom of the screen



Vault

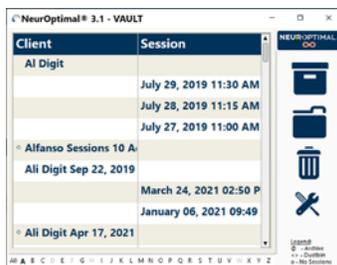

Media Player




3

Connect USB cable to zAmp and computer or tablet.




4

Apply sensors.

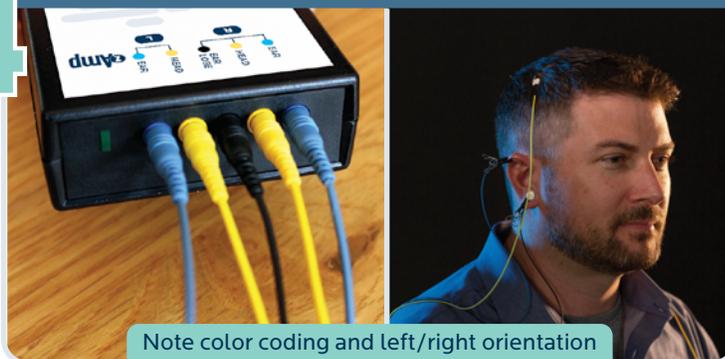


5

Double tap "Client" to add your name (if needed).

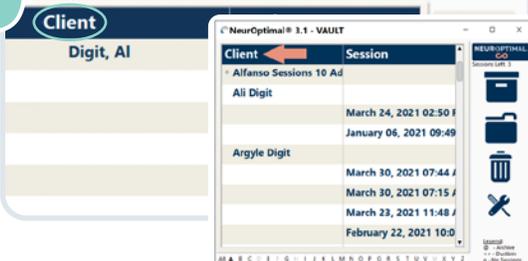

6

Double tap your name to start a Session.

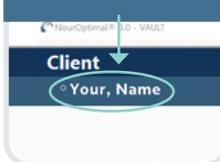

IMPORTANT
Please always have your system connected to the Internet.

Please contact your rental provider at _____ with any questions.