


ZenConnect


Disk Cleanup Guide

This guide will explain how to perform a disk cleanup. Files can add up over time, and maintaining your system through regular cleanups will ensure you always utilize the space on your drive effectively without waste.


- 1 Open the Start Panel. Right click **Disk Cleanup** and run it as administrator.

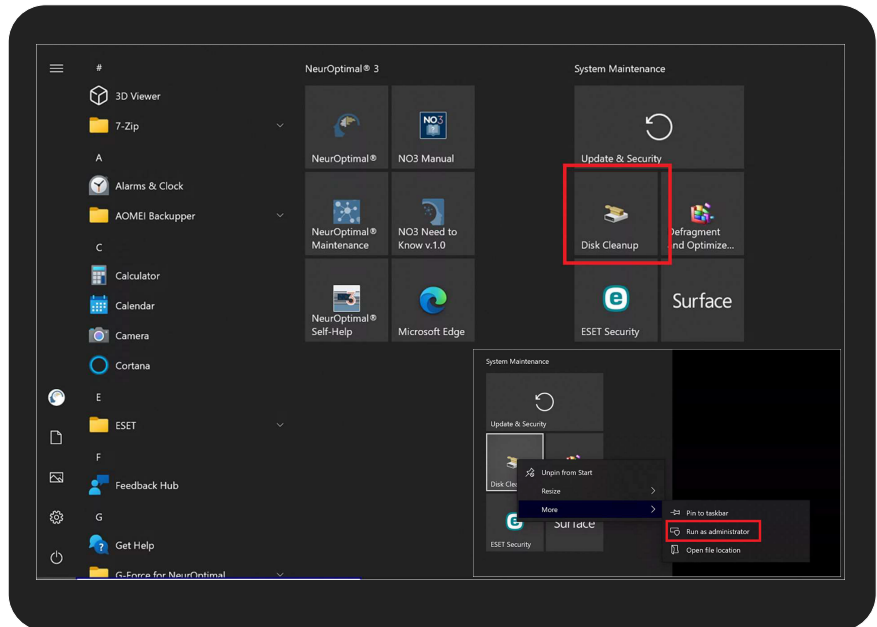


- 2 Select **OS (C:)**. PHOENIX (D:) is the Zengar[®] recovery partition and can be ignored.

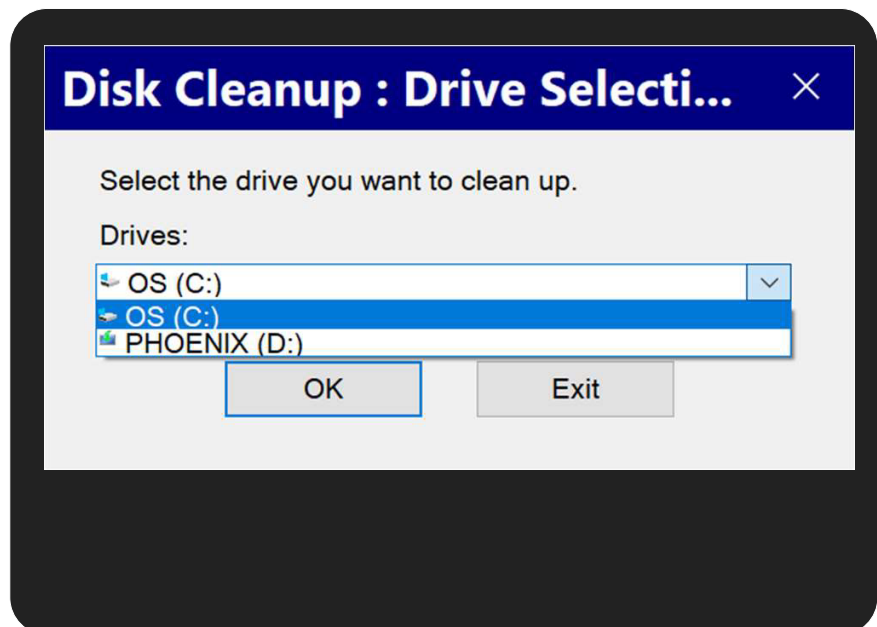

3a. Select all items in the list.

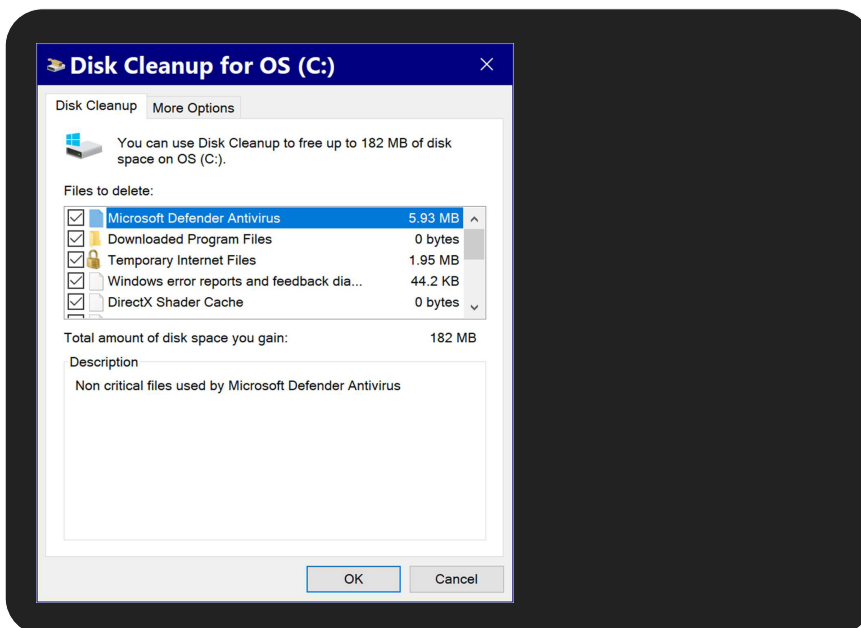


3b. Select all items in the list.

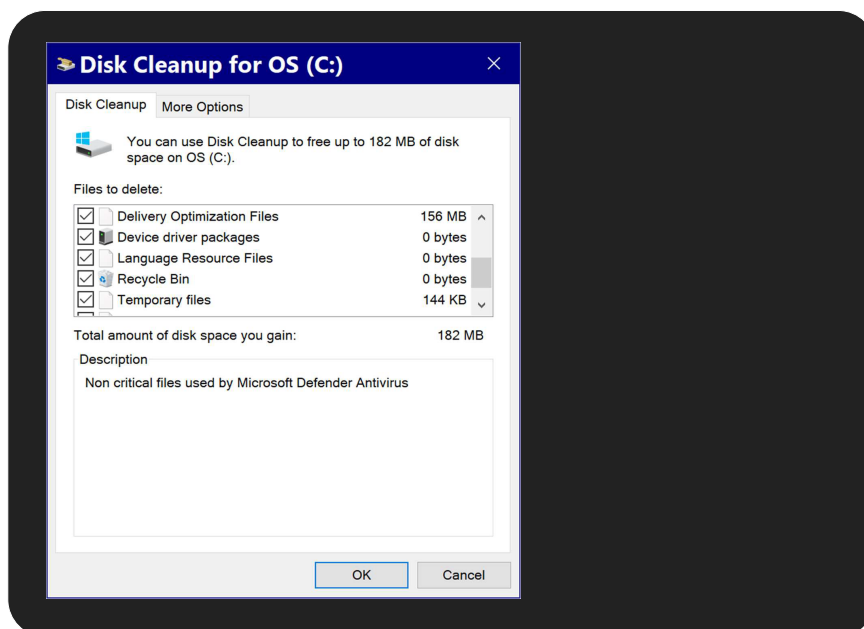

3c.

Select all items in the list.

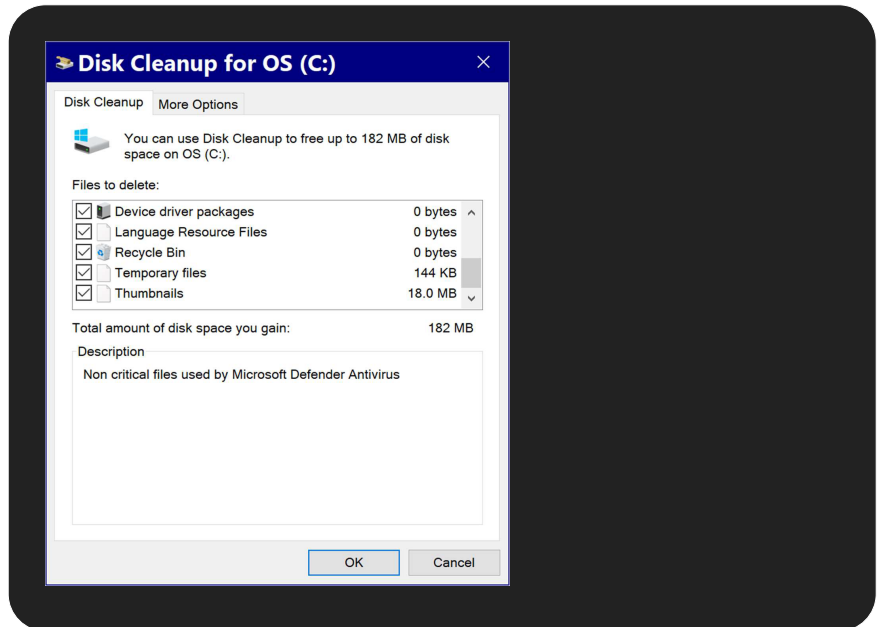

4

After selecting all items, open the **More Options** tab. Click **Clean up...** Then press **OK**.

Disk Cleanup will now remove all the files selected and delete old restore points.

