


ZenConnect


G-Force Configuration

This guide will show you how to configure G-Force and Sound Settings.


- 1 Click on the Windows icon.

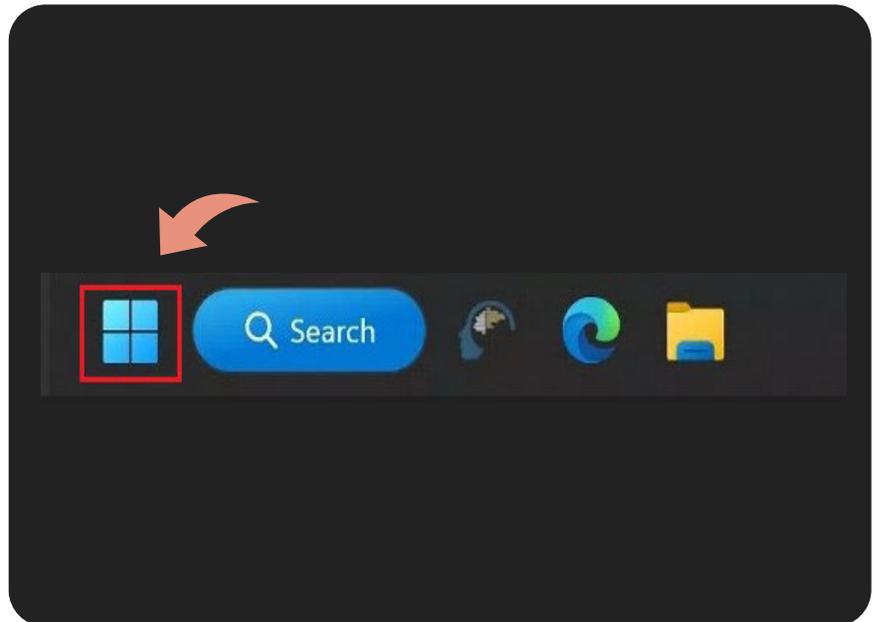


- 2 Type **Sound Settings** in the search bar and press **Enter** on your keyboard or click **Sound Settings** under **Best match**.

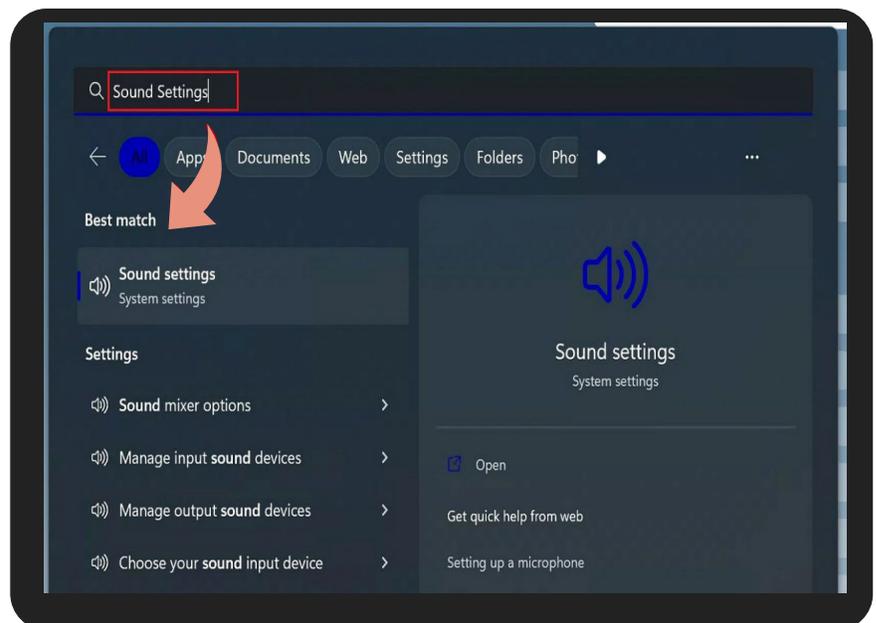

3a. *Windows 10.*
When the **Sound** window opens, click **Sound Control Panel** and then proceed to step 4.

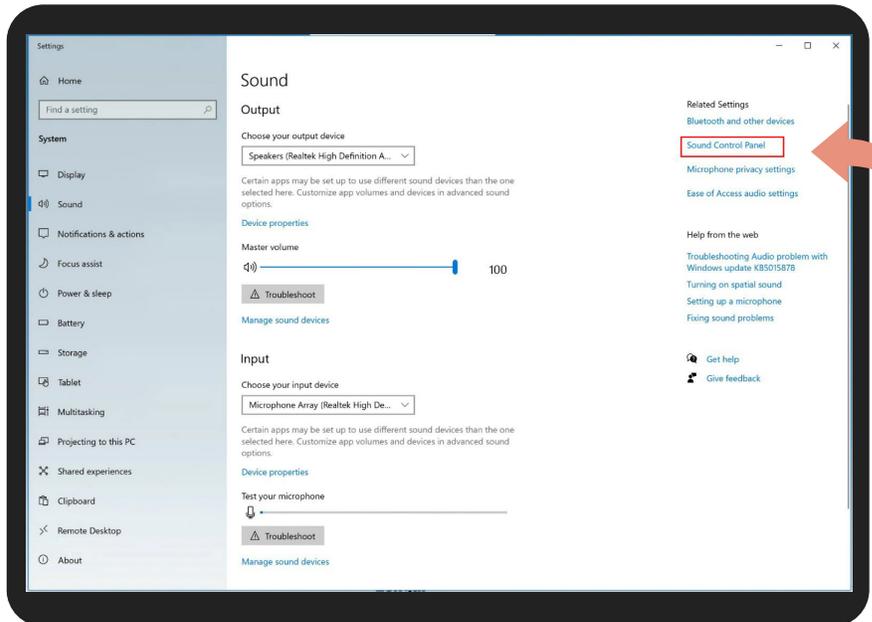


3b. *Windows 11.*
When the **Sound** window opens, scroll down the page and click **More sound settings** and proceed to step 4.

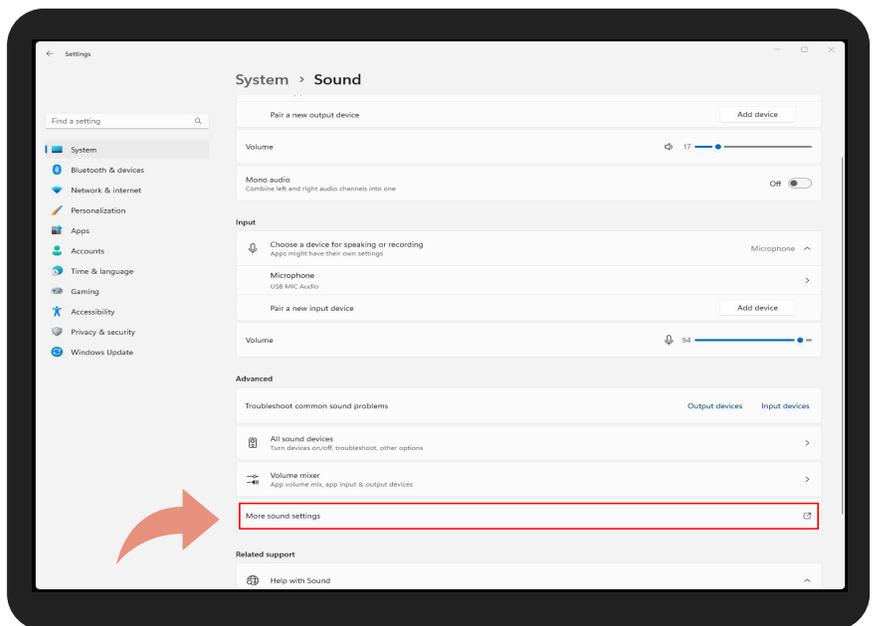


4

Select the **Playback** tab and verify **Speakers** are set as **Default Device**.

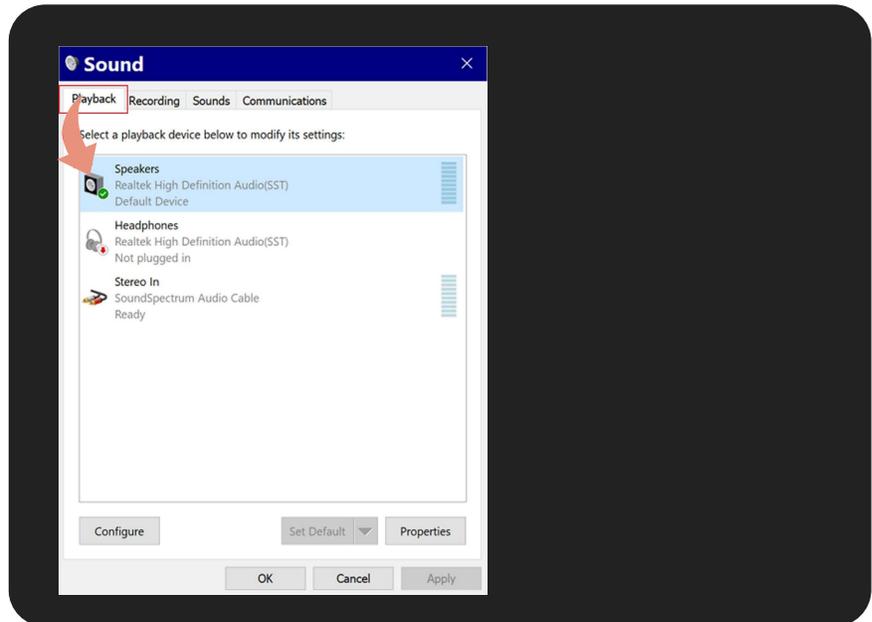

5

- In the same **Sound** window, select the **Recording** tab.
- Right-click **Stereo Out** and select **Properties**.
- In the next window, select the **Listen** tab.
- Check **Listen to this device**.
- Use the drop-down menu and set it to **Default Playback Device**.
- Press **OK** on both windows.

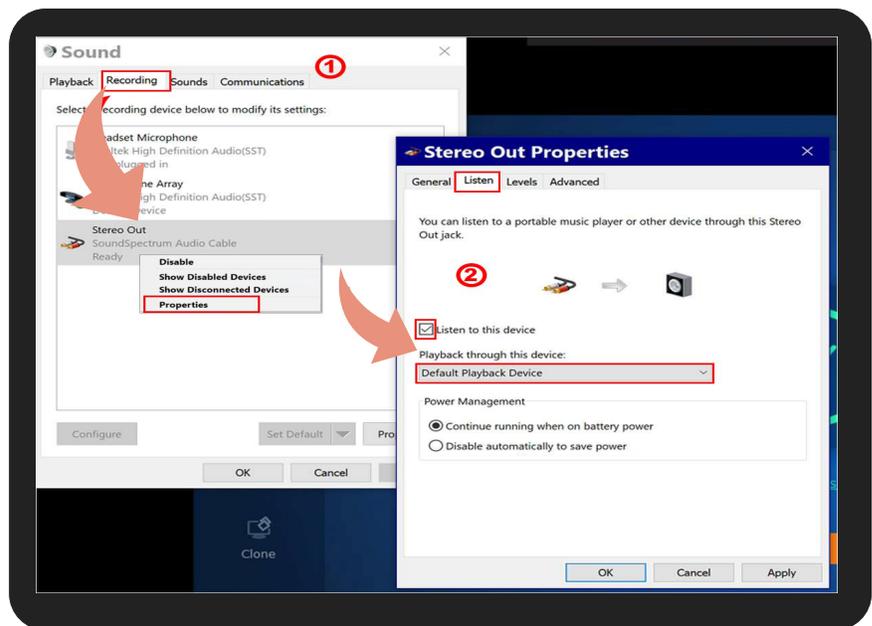

6

Right click the G-Force screen and select **Stereo Out (SoundSpectrum Audio Cable)**

