

# Reactivate G-Force



This guide shows you how to reactivate G-Force using the "Reactivate G-Force" menu option in NeurOptimal®

## Reactivate G-Force

1 Launch NeurOptimal®

2 Go to Tools menu and select "Reactivate G-Force"

The screenshot shows the NeurOptimal 3.5 - VAULT application window. The main window contains a table with the following data:


Client	Session
Digit, AI	
	July 29, 2019 02:30 PM
	July 28, 2019 02:15 PM
	July 27, 2019 02:00 PM

Below the table is a keyboard layout with letters A through Z. To the right of the table is a vertical toolbar with icons for a folder, a trash can, and a wrench. A context menu is open over the wrench icon, listing the following options:


- Session
- zAmp Tests...
- Preferences
- Renew PASS
- ZenConnect
- Refresh Licensing
- Restart NO Media Player
- Reactivate G-Force**
- Cleanup VAULT
- How To Videos
- Show Context Help (Ctrl+H)
- About...

At the bottom of the window is a media player control bar with a play button, a volume icon, and a timer showing 00:00 / 00:00.

- 2 If the "Reactivate G-Force" menu option is greyed out, go to Preferences and the NOMP tab and make sure "Enable G-Force" is checked

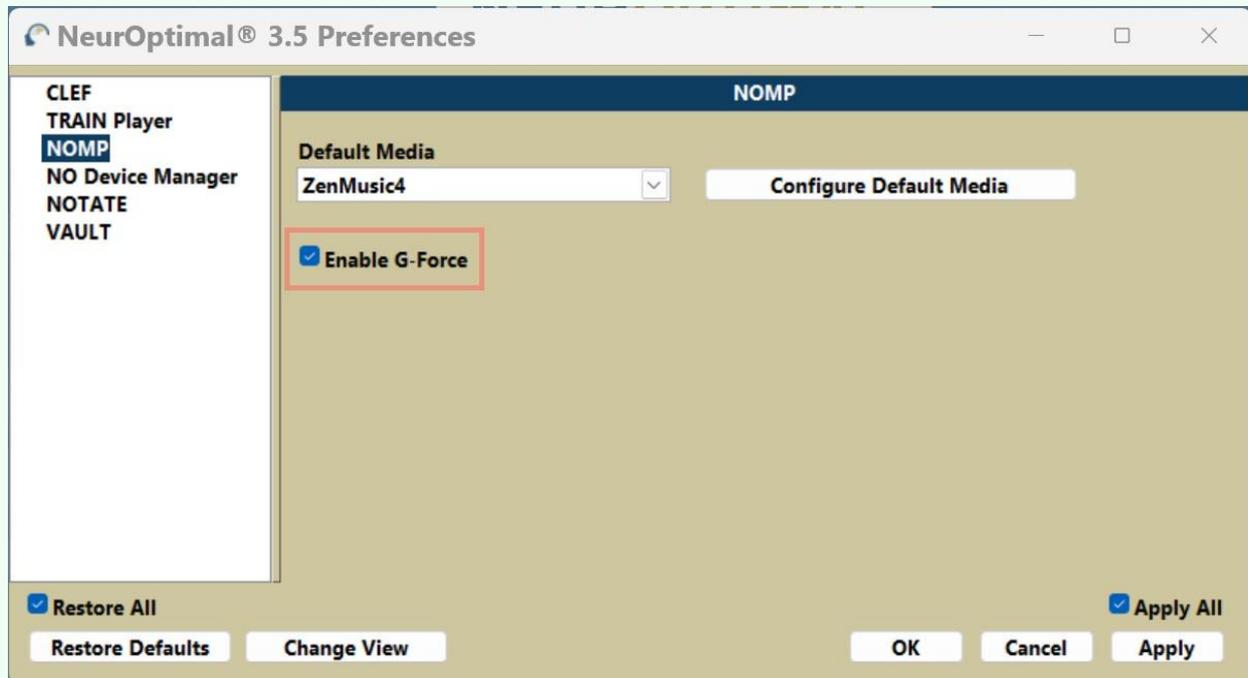

- 3 Click Continue in the dialogue box

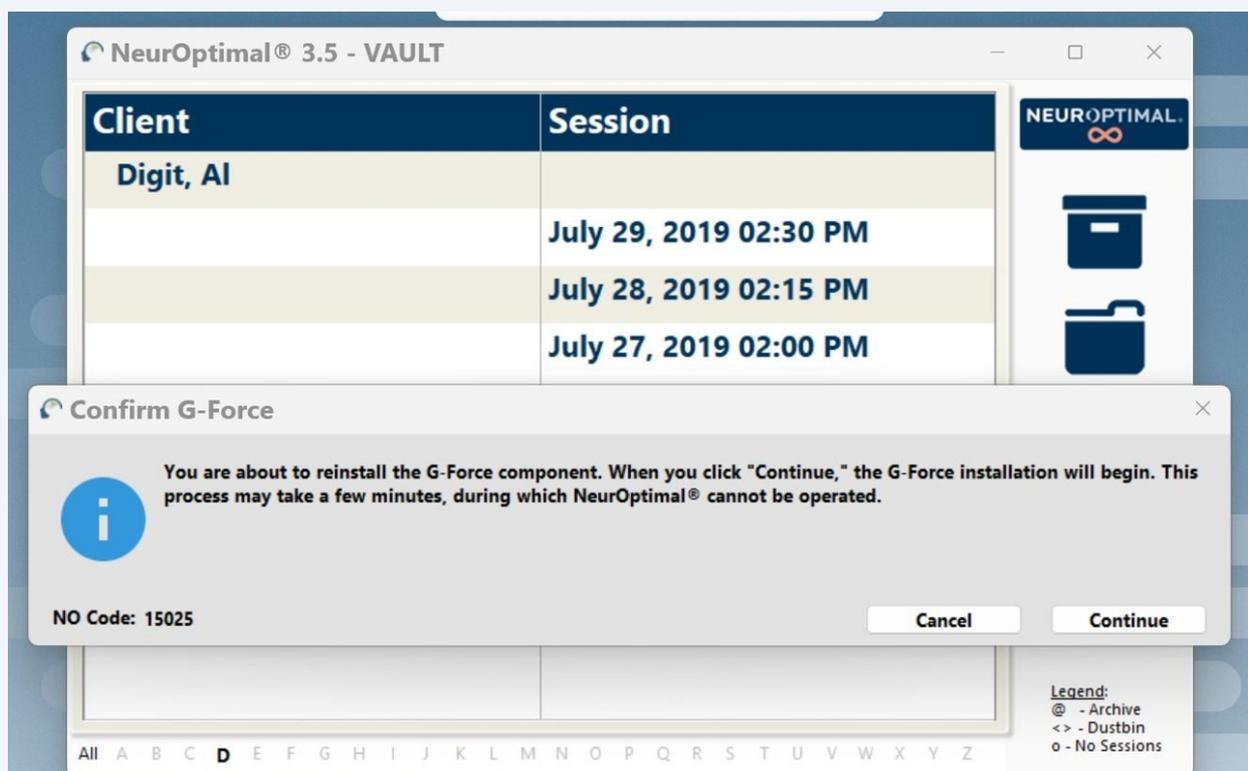

4 Select Yes when asked if you want to allow this app to make changes

User Account Control



Do you want to allow this app to make changes to your device?



installerbase


Verified publisher: Zengar Institute Inc.  
File origin: Hard drive on this computer

[Show more details](#)

Yes

No

5 Let the process finish

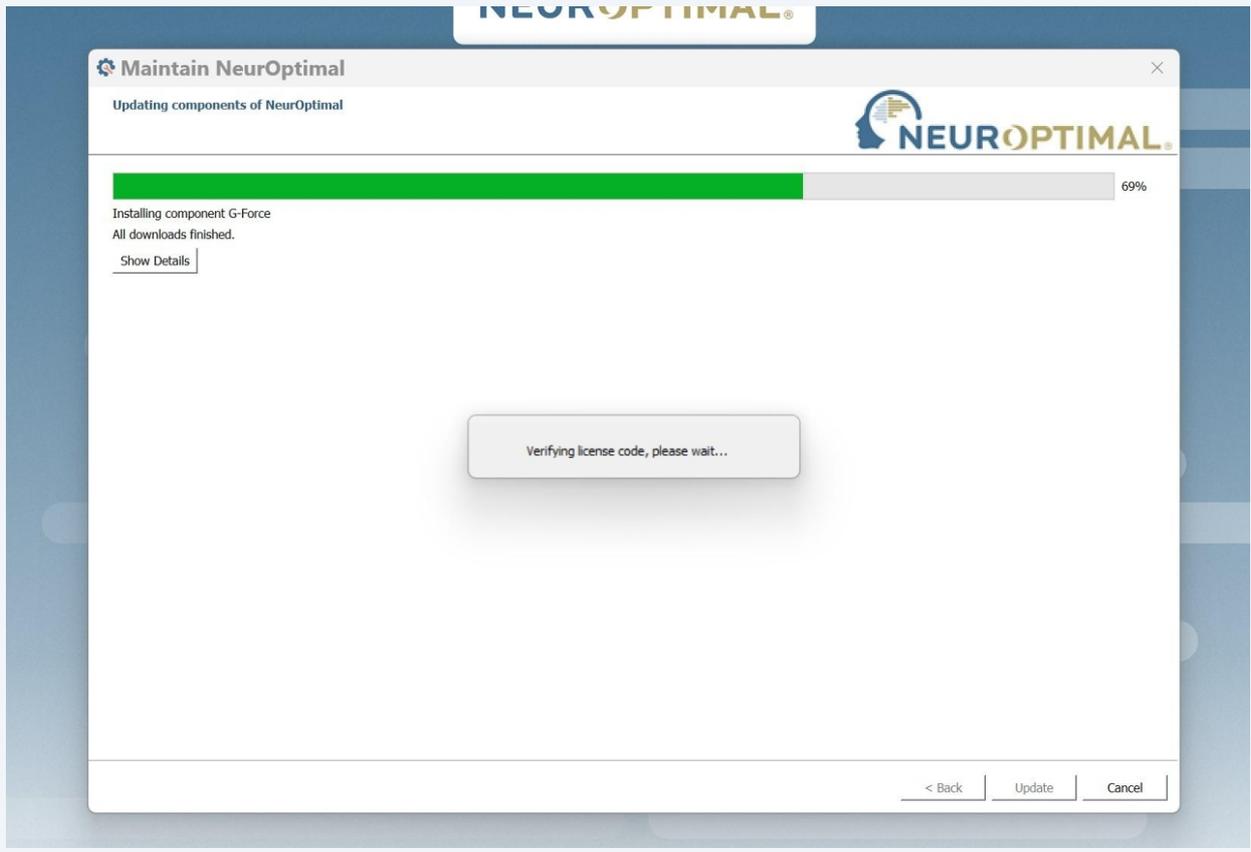

6

G-Force pops up and you are taken back to the VAULT, ready to start a new Session

